

Max patient weight 455kg/72 stone

Control Unit

(1) Power Switch

Used to turn the power on and off.

(2) Standby

When light is on, press the Power button and set preferred mode of therapy and comfort level.

(3) Back Raise Sensor

If the head of the mattress is raised above 25 degrees, the LED will turn on and the system will increase the pressure in the mattress to compensate for increased pressure in the torso section.

(4) Comfort Adjust Buttons

The Comfort Adjust is located on the right side of the control panel. The system can be customised to meet individual patient needs within a therapeutic window. These buttons allow you to control the firmness on the Arise 1000. Use (+) to increase or (-) to decrease firmness. **DO NOT USE THESE BUTTONS TO AUTOFIRM/P MAX THE PATIENT.**

(5) Alarm Rest

If power fails to the Arise 1000, the control unit will alarm audibly and the indicator light will illuminate. To cancel the alarm press Alarm rest.

Lockout Feature (Control unit not responding)

Five minutes after the last mode adjustment, the Power light begins to flash indicating Lockout is enabled. You must release the lockout in order to change the therapy mode. Unlock by pressing both Comfort Adjust arrows simultaneously.

CPR



Hold the controller steady with one hand. Pinch here on both sides with other hand and pull firmly

The hose connection at the control unit is marked CPR. Disconnect the hose from the control unit by pinching both sides of the connector at the point shown. To resume therapy, reattach the hose to the control unit.

Modes

Autofirm



To quickly inflate the mattress to maximum pressure, use Autofirm mode. This mode may be desirable to assist with patient transfers and other nursing procedures. Therapy system will automatically return to the Static/Floatation mode after 12 minutes.

Therapy



In this mode, the Arise 1000 mattress provides full body floatation and remains static, allowing for patient immersion and envelopment.

Pulsate



Pulsate will slightly decrease the pressure in all therapy cells every 30 seconds, which some patients find comforting.

Width & Length Extension, Bolsters

The Arise 1000 can be width and length extended to fit the extended Bari 10A platform.

Width extension

To inflate the width extension, turn the upper left and/or right knobs to 'inflate'.

Length extension

To inflate the foot extension, turn the lower left knob to 'inflate'.

Cair Rails (bolsters)

To inflate the bolsters, turn the lower right knob to 'inflate'.

