

## Versal Plus Pack - Sling and Slide Sheet

### Single Patient Use

### 100cm x 230cm & 150cm x 230cm

The Versal Plus is a combination pack consisting of the Versal Slide Sheet & Banana Repositioning Sling to offer a safer more dignified experience for plus sized patients when moving and handling. The pack reduces the effort and reduces the manual handling risk of the staff undertaking these manoeuvres. Prior to the use of a Slide Sheet and sling a thorough risk assessment in line with local policy should be undertaken to identify and mitigate against potential risks. Principles of independent movement should be considered wherever possible (e.g. whether a patient can assist in the movement using the slide sheet).

#### Inspect Slide Sheet before each use.

- Ensure that the correct Slide Sheet is being used.
- Do not use Slide Sheet to lift patient.
- Do not use if Slide Sheet is worn, damaged or wet.

**SWL:** 454kg Sling

**Cleaning:** Do not launder. Discard of the device in line with local hospital protocol after single patient use.

#### **WARNING – It is recommended to conduct a specific patient and scenario risk assessment when using Versal Plus**

- Always use appropriate equipment and sizes.
- Carry out full Risk Assessment.
- Follow RCN (or equivalent) guidelines.

**FR** Pack Versal Plus - Sangle et drap de repositionnement

**IT** Versal Plus Pack - Telo di posizionamento con cinghie

**ES** Pack Versal Plus - Eslinga y sábana de transferencia

**DE** Versal Plus Pack – Gurt und Gleitmatte

**DK** Versal Plus Pack - Slynge og glidelagen

**NO** Versal Pluss-pakke – Bind og glidelaken

**SE** Versal Plus Pack - Lyftskynke och förflyttningsmatta

**NL** Versal Plus Pack - Draagband en glijlaken



**GBUK Group Ltd.**

Woodland House  
Blackwood Hall Business Park  
North Duffield, Selby  
North Yorkshire  
YO8 5DD, UK

EU: +44 (0) 1757 288 587  
USA: 18665 289 422



[www.gbukbanana.com](http://www.gbukbanana.com)

Symbol glossary is available at [www.gbukbanana.com](http://www.gbukbanana.com)

B-VersalPlus-03

2007

11