

IMPORTANT SAFETY INFORMATION: DO NOT REMOVE

Quick User Guide: Bari 10A Bariatric Four Section Electric Profiling Bed Frame



Max safe working load 477kg / 75st


BED OPERATION

1. Press to raise back
2. Press to lower back
3. Press to raise knee
4. Press to lower knee
5. Press to raise bed
6. Press to lower bed
7. Press for Comfort Chair Position
8. Press button to lower both back and knee to flat and place bed at low height
9. Press for reverse Trendelenburg (foot down tilt)
10. Press for Trendelenburg (head down tilt)
11. Press to light up control panel
12. Press to lock out a function, in conjunction with pressing function



Hand Control

Integrated Control Panel

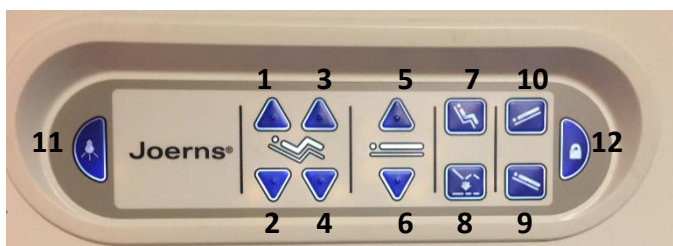

FUNCTION LOCK-OUT

To lock-out functions, press the padlock symbol plus the required function on the integrated control panel (located on the foot board). This will automatically lock this function on the hand control. To reverse, press the padlock plus the function you wish to unlock. **NOTE: As a safety feature the control system will automatically lock-out the knee-down function if the platform is in foot down tilt (bed is in Comfort Chair position or reverse Trendelenburg position).**

To unlock, return the platform to flat using the head down tilt (Trendelenburg) function (10). The bed will pause at flat. All functions will now unlock.



For support, contact Five Mobility T: 0800 193 2523
E: info@fivemobility.co.uk, W: www.fivemobility.co.uk

Quick User Guide: Bari 10A Bariatric Four Section Electric Profiling Bed Frame

OPERATING THE BRAKE

There is one brake castor and one steer castor located at both the head and foot of the bed.

To apply the brakes, push down the red pedal on each of the castors at both the head and foot end.



USING THE STEER LOCK FUNCTION TO ASSIST WITH MOVING THE BED

To engage the steer, ensure both the red brake castors are disengaged.

Then push the green steer pedal down on either one of both the castors. Should only one steer castor be applied please ensure this is the opposite end from which bed is being steered.



EXTENSION OPERATION



Length Extension: Slide the frame extender out by pulling the latch knob on the side of the bed frame. Next, slide the deck extender by pulling the latch knob underneath the deck and sliding the extender out.

Width Extension: Pull purple latch handle along the bottom edge of each section and pull.

ADDITIONAL FEATURES



CPR Release:
Rotate red handle up and hold until deck is flat. Located at foot end of bed.



Angle Indicators:
The angle indicator at the head end allows caregivers to read the head of bed angle. The indicator at the foot end allows the Trend/reverse Trend angle to be viewed.



Side Rail Release:
Pull purple handle towards you and rotate side rail down.