

# Slide

## Product Guide



**Banana**  
Part of the GBUK Group ●●●●

Please read this guide before use.

EGF & EGT

## Slide Sheets

### Single Patient Use

The intended use of a Slide Sheet is to reduce the friction and shear during the task of moving and /or repositioning a patient on a surface; it also reduces the effort and reduces the manual handling risk of the staff undertaking these manoeuvres (Fray 2015). Prior to the use of a Slide Sheet a thorough risk assessment in line with local policy should be undertaken to identify and mitigate against potential risks. Principles of independent movement should be considered wherever possible (e.g. whether a patient can assist in the movement using the slide sheet).

#### Inspect Slide Sheet before each use.

- Ensure that the correct Slide Sheet is being used.
- Do not use Slide Sheet to lift patient.
- Do not use if Slide Sheet is worn, damaged or wet.

**Cleaning:** Do not launder. Discard of the device in line with local hospital protocol after single patient use.

**Tumble Dry:** Do not tumble dry.

**Tearing Strength:** ISO 13937.2:2000 Single-shaped method – Warp 65n and Weft 63n.

**Tensile Strength:** ISO 13934.1:2013 Strip Test – Warp 560n and Weft 400n

**Disposal:** Dispose of device as per hospital policy.

#### **WARNING – It is recommended to conduct a specific patient and scenario risk assessment when using a Slide Sheet**

- Always use appropriate equipment and sizes.
- Carry out full Risk Assessment.
- Follow RCN (or equivalent) guidelines.

GB	Flat Sheet with Handles	Flat Sheet no Handles	Tubular Sheet
FR	Drap de repositionnement plat avec poignées	Drap de repositionnement plat sans poignée	Drap de repositionnement tubulaire
IT	Telo piatto con maniglie	Telo piatto senza maniglie	Telo di posizionamento tubolare
ES	Sábana de transferencia plana con manijas	Sábana de transferencia plana sin asas	Sábana de transferencia tubular
DE	Flache Gleitmatte mit Handgriffen	Flache Gleitmatte ohne Handgriffe	Röhrenförmige Gleitmatte
DK	Fladt glidelagen med håndtag	Fladt glidelagen uden håndtag	Rørformet glidelagen
NO	Flatt glidelaken med håndtak	Flatt glidelaken uten håndtak	Rørformet glidelaken
SE	Platt förflyttningsmatta med handtag	Platt förflyttningsmatta utan handtag	Tubformig förflyttningsmatta
NL	Vlak schuiflaken met handgrepen	Vlak schuiflaken zonder handvatten	Buisvormig schuiflaken



**GBUK Group Ltd.**  
Woodland House  
Blackwood Hall Business Park  
North Duffield, Selby  
North Yorkshire  
YO8 5DD, UK

[www.gbukbanana.com](http://www.gbukbanana.com)

EU: +44 (0) 1757 288 587  
USA: 18665 289 422



Symbol glossary is available at [www.gbukbanana.com](http://www.gbukbanana.com)

# Slide

## Product Guide



**Banana**  
Part of the GBUK Group ●●●●

Please read this guide before use.

EGF & EGT

Product Code	Width (W)	Length (L)	SWL
--------------	-----------	------------	-----

### Flat Sheets with Handles

EGF072200H	72cm	200cm	225kg
EGF090200H	90cm	200cm	225kg
EGF100100H	100cm	100cm	225kg
EGF100150H	100cm	150cm	225kg
EGF140200H	140cm	200cm	225kg

### Flat Sheets without Handles



EGF072200	72cm	200cm	N/A
EGF090200	90cm	200cm	N/A
EGF100100	100cm	100cm	N/A
EGF100150	100cm	150cm	N/A
EGF100200	100cm	200cm	N/A
EGF140200	140cm	200cm	N/A

### Pack of 2 Flat Sheets without Handles

B-EGF072200P	72cm	200cm	N/A
B-EGF100200P	100cm	200cm	N/A
B-EGF140200P	140cm	200cm	N/A

### Tubular Sheets

EGT072072	72cm	72cm	N/A
EGT072120	72cm	120cm	N/A
EGT072150	72cm	150cm	N/A
EGT072200	72cm	200cm	N/A
EGT100100	100cm	100cm	N/A
EGT100120	100cm	120cm	N/A
EGT100150	100cm	150cm	N/A
EGT100200	100cm	200cm	N/A
EGT100230	100cm	230cm	N/A
EGT140200	140cm	200cm	N/A

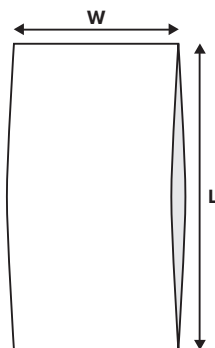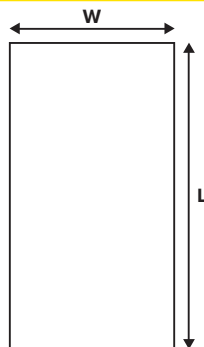

**GBUK Group Ltd.**  
Woodland House  
Blackwood Hall Business Park  
North Duffield, Selby  
North Yorkshire  
YO8 5DD, UK

[www.gbukbanana.com](http://www.gbukbanana.com)

EU: +44 (0) 1757 288 587  
USA: 18665 289 422



**CE**

Symbol glossary is available at [www.gbukbanana.com](http://www.gbukbanana.com)

EGF & EGT-03

2007  
12